MAHLE

FFP2 NR Schutzmaske

FFP2 NR Protective mask

Masque de protection FFP2 NR

Mascarilla FFP2 NR

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Maska ochronna FFP2 NR

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Made in Austria
EN 149:2001+A1:2009

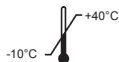
End of storage period

See the manufacturer's information leaflet

Maximum relative humidity of storage conditions

Temperature range of storage conditions

Notified body:
DEKRA Testing and Certification GmbH
Handwerkstr. 15, 70565 Stuttgart
Germany

For more information, see
mahle-aftermarket.com.

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EN

Instructions for use:

- Before entering a hazardous area, always make sure that the mask fits tightly against the face.
- If breathing becomes more difficult or the mask becomes contaminated, it must be replaced immediately.
- The mask is designed for single use only—do not mend, clean, or modify.
- The mask does not produce oxygen and may only be used in an environment with an oxygen content of at least 19 %.
- Provides no protection against gases and vapors that are harmful to health.
- It is unlikely that leakage requirements will be met if there is facial hair underneath the seal line on the face.
- Do not use the mask in explosive atmospheres

Storage conditions:

- Protect from direct sunlight and cold.
- Protect against exposure to fire, rain, acids, and alkalis.

The NR mask is not reusable and must not be worn on consecutive shifts.

Instructions for putting on the mask

1. Unfold the mask fully. Check the position of the nose clip.
- 2a. Place the mask on the chin so that the mouth and nose are completely covered. Pull ear straps behind the ears and position them there. The straps are elastic and therefore self-adjusting.
- 2b. In addition to step 2a, the enclosed ear strap extension can be used. If the leakage or fit test (step 4) fails, the tightness of the ear straps must be adjusted.
3. Use both hands to adjust the shape and fit of the nose clip. Adjusting the clip prevents leakage during use. Adjusting with only one hand may affect the correct fit of the mask.
4. Before using the mask, check that it fits tightly against the face.
- 4.1 Carefully cover the mask with both hands without changing the fit.
- 4.2 Then breathe out forcefully.
- 4.3 If there is a leak in the nose area, readjust the nose clip. Repeat fit test.
- 4.4 If there is a leak at the edge of the mask, check and adjust the fit of the straps. Repeat fit test.

Tips to ensure the mask fits tightly

To achieve an adequately tight fit, it may be necessary to shorten the straps by tying a knot. In particular, if there is insufficient contact pressure against the face when the mask is put on (e.g., there is an obvious gap between the mask and the face) and/or subjective leakage is perceived when breathing in.

Caution: Tying knots in the straps generally increases the likelihood of pain occurring around the ears. If pain persists, choose a mask model that is suitable for you or use an ear strap extension that sits across the back of your head.

Because face shapes vary greatly, the masks may not be suitable for everyone.

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